Zoff
Eye Performance

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Glasses User Guide

Thank you very much for purchasing our product. This guide is intended to help you use the glasses safely and comfortably. Please be sure to read this guide carefully before using the glasses. Parents or guardians should explain to their children on how to use the glasses. Please keep this user guide in a safe place.

Precautions

- To ensure safe and comfortable use of your glasses, please be sure to have a periodic inspection once a year at your local Zoff including visual acuity measurement, fit adjustment, inspection and replacement of consumable parts, and lens and frame deterioration.
 - This user guide is intended to help you use the glasses correctly and safely. Although we have taken great care in preparing this user guide, please contact your local Zoff if you have any questions or concerns.
 - The contents of this user guide are subject to change without notice for improvement.
-

Warning

Improper handling may result in serious consequences including blindness or severe injury.

Eyeglass frames or lenses may break or crack with high impact. This may cause serious injuries to the face or eyes. Broken pieces and other objects can cause blindness if they get into the eyes. Special care should be taken when using glasses during sports activities.

Do not place the glasses near a heater or fire. Glasses made of celluloid material will start to burn at 170°C which may cause a fire or burns.

Avoid using colored lenses in dark places. Colored lenses with a density of 25% or more and a visible light transmittance (VLT) of 75% or less should not be used for driving at dusk or at night. Insufficient light intensity can lead to decreased vision and is dangerous.

Do not look directly at strong light such as the sun or welding regardless of the density of the colored lens. It may cause serious eye injury or blindness.

Caution

Improper handling may result in injury or material damage.

Do not use glasses with loose screws, deformed frames, or scratched or cracked lenses, and have them inspected at your local Zoff.

Please get used to your new glasses before driving or operating a car. Be especially careful when riding bicycles, motorcycles, and driving cars with progressive multi-focal lenses. When you are not used to using the lenses, be careful when going up and down the stairs or places with difference in level as you may feel swaying and distortion on the sides of the lens or have difficulty seeing the perspective.

If you notice any abnormality in vision due to scratches or fogging of the lens, stop using immediately and consult your local Zoff.

Degradation of the frame over time may cause exposing of material and discoloration from peeling and cracking of surface layer. Continued use in this state may cause metal allergies. Please have your glasses inspected at your local Zoff about once a year.

UV-cut processed lenses can protect against normal harmful UV rays, but they cannot be used as UV protection for commercial use.

Do not use the glasses in places with high temperature such as a sauna room, or a place with low temperature such as a freezer. Deformation of the frames or lenses may interfere with vision or cause the lenses to detach from the frames.

Half-rim frames, in which the lens is grooved and the nylon thread is passed through to hold the lens in place, the nylon thread may loosen or break due to degradation over time, causing the lens to detach. In addition, the nylon thread can also break due to heat or impact. Please be careful not to place near fire or subject to impact. Please have your glasses inspected about once a year and have the nylon thread replaced as needed.

If you experience any abnormalities such as itching, rashes, or eczema where the frame touches your skin, stop using the glasses immediately and seek medical attention from a dermatologist. Along with treatment, it is necessary to replace the parts or frame that suit your body constitution.

Glasses that are distorted or do not fit properly will interfere with your vision and can cause an accident or injury. Please have your glasses adjusted at your local Zoff before use.

If the glasses do not fit your face well, you may experience a rash on your nose or behind your ears. If you notice any abnormalities, please consult your doctor or local Zoff.

Since rimless types (two-point) are weak in strength than rimmed frames, special care must be taken when handling and storing. Do not use during intense sports activities as it is dangerous. Also, do not forcibly spread or twist. Please consult with your local Zoff if you need to adjust the fit, or if you notice any deformation or other abnormalities.

Handling Precautions

- When putting on and taking off the glasses, use both hands to hold the temples. Putting on and taking off with one hand may cause glasses to lose its shape.
- Rest the glasses with lenses facing up. Resting the glasses facing down can leave scratches on the lens.
- Do not intentionally alter or scratch the glasses. Doing so may cause damage.
- Exposure of both frame and lens to high temperature (60°C or above) or sudden change in temperature could cause deformation, surface layer separation or cracking. Please be careful not to leave the glasses in a car under the blazing sun, on the beach, etc.
- To clean your glasses, first wash them with water then wipe using tissue paper or soft cloth. Please note that dry wiping may cause scratches.
- Frame's nose pads can be cleaned by washing with soft bristle toothbrush.
- When wiping the lens, hold the outer side of the lens to be wiped and wipe gently. Wiping with excessive force may cause the frame to lose its shape or be damaged.
- In case the glasses are heavily soiled, wash with diluted neutral detergent. After washing, rinse thoroughly with water and wipe dry. If the lens is left wet, it will become stained and cannot be removed. It can also cause damage to the coating film.
- Do not use alkaline detergent such as soap and hand soap for washing.

- Do not use benzine or paint thinner for cleaning. It may cause deterioration, discoloration or degradation of the plastic and paint.

Zoff SMART Handling Precautions

- Please do not twist or bend the frame in its finished product form with the lenses installed.
- Extreme bending of the frame may cause damage. Also, there is a risk of injury from the lens coming off with great force.

Zoff NIGHT&DAY Handling Precautions

- Please stop using this product if you feel any discomfort.
- This product uses magnets. Please be careful not to let the magnet part come in direct contact with any area above your neck.
- Lenses on the clip-on cannot be changed regardless of whether they are prescription or non-prescription.
- Please do not use this product for extended period of time.
- The degree of polarization differs depending on lens type.
- Since the clip-on is attached to the frame body using magnets, please be careful of strong wind, applying too much force, twisting, etc. as these can cause the clip-on to detach from the frame body.
- If the visible light transmittance (VLT) is less than 75%, driving and use on the street are prohibited at night.
- If the visible light transmittance (VLT) is 8% or less, driving and use on the street are prohibited regardless of day or night.

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